


medela


Magic InBra™

Scan to watch how to use videos

Scannen Sie, um Anleitungsvideos anzusehen
 Scannez pour regarder les vidéos explicatives
 Escanea para ver vídeos explicativos
 Scan om instructievideo's te bekijken
 Scansiona per guardare i video dimostrativi
 Escaneie para assistir aos vídeos de explicativos

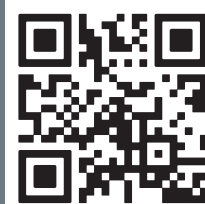

Before First Use • Vor der ersten Verwendung • Avant la première utilisation • Antes del primer uso •
 Vóór het eerste gebruik • Prima del primo utilizzo • Antes da primeira utilização

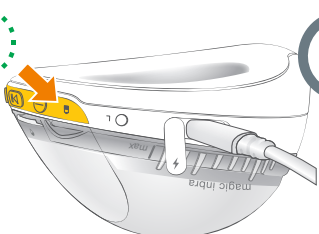

60'

Charge for 1 hour

1 Stunde aufladen
 Chargez pendant 1 heure
 Cargar por 1 hora
 1 uur opladen
 Carica per 1 ora
 Carregar durante 1 hora

5'





Boil (except motor)

Abkochen (außer Motoreinheit)
 Faire bouillir (sauf l'unité moteur)
 Hervir (excepto la unidad del motor)
 Koken (behalve het motorgedeelte)
 Bollire (eccetto l'unità motore)
 Ferver (exceto o motor)

Rotate to disassemble

Zum Auseinanderbauen drehen
 Tournez pour démonter
 Girar para desmontar
 Draai om te demonteren
 Ruotare per smontare
 Girar para desmontar

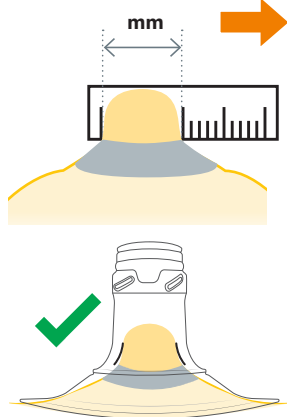
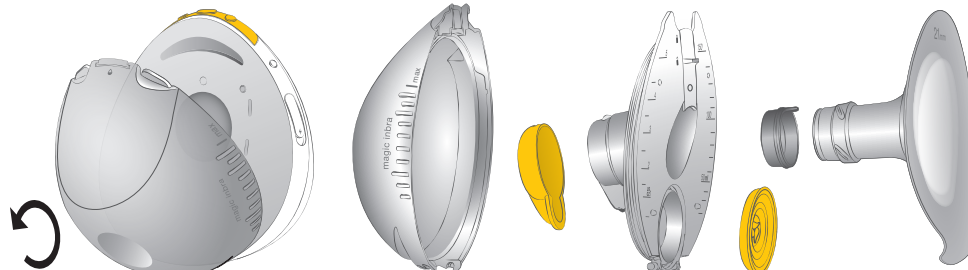


<14 mm	15 mm, 16 mm, 17 mm	18 mm, 19 mm, 20 mm	21 mm, 22 mm, 23 mm


For best milk output and comfort choose the right size.

Für optimale Milchmenge und Komfort wählen Sie die richtige Größe.
 Pour un confort optimal et une production de lait maximale, choisissez la taille adaptée.
 Para una mejor extracción y comodidad, elige el tamaño correcto.
 Voor de beste melkproductie en comfort, kies de juiste maat.
 Per una migliore produzione di latte e comfort, scegli la misura giusta.
 Para uma extração de leite mais eficaz e confortável, escolha o tamanho certo.


Check assembly • Zusammenbau überprüfen • Vérifiez l'assemblage • Verifica el ensamblaje •
 Controleer de montage • Controlla l'assemblaggio • Verificar a montagem

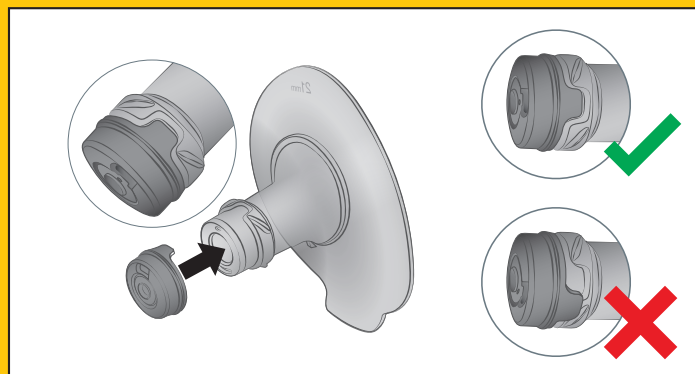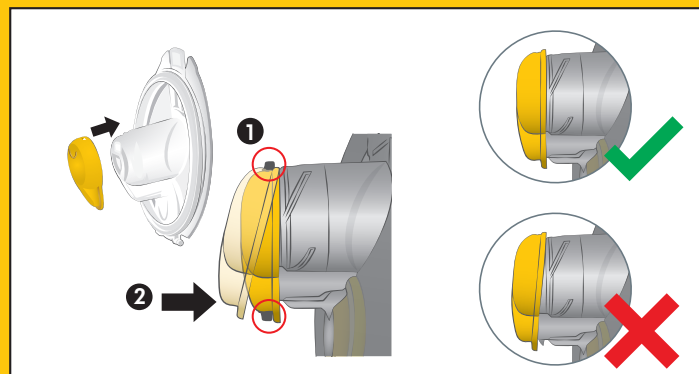

Check assembly • Zusammenbau überprüfen • Vérifiez l'assemblage • Verifica el ensamblaje • Controleer de montage • Controlla l'assemblaggio • Verificar a montagem

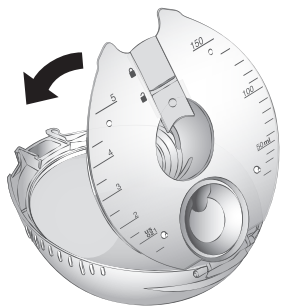

Close • Schließen • Fermer •
Cerrar • Sluiten • Chiudi •
Fechar

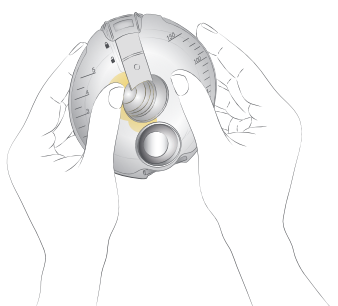


Push firmly • Fest drücken • Appuyez fermement •
Presionar firmemente • Stevig aanduwen • Premi
con decisione • Pressionar firmemente




Close • Schließen • Fermer •
Cerrar • Sluiten • Chiudi •
Fechar

After Pumping • Nach dem Abpumpen von Milch • Après l'expression du lait • Después de la extracción de leche • Na het kolven • Dopo l'estrazione del latte • Após a extração de leite

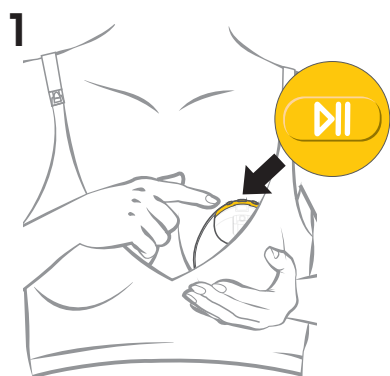


Pause
Pausieren
Pause
Pausa
Pauzeer
Pausa
Pausa

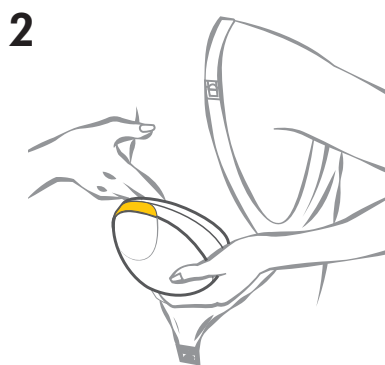


Lean forward & break latch

Nach vorne lehnen und das
Vakuum lösen
Penchez-vous vers l'avant et
rompez la prise
Inclínate hacia adelante y rompe
la succión
Leun naar voren en verbreek het
vacuüm
Piegati in avanti e stacca la
suzione
Incline-se para a frente e solte a
vedação

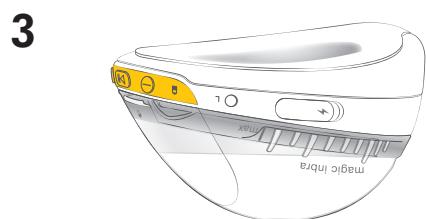


Place on a flat surface • Auf eine flache Oberflä-
che legen • Placez sur une surface plane •
Coloca en una superficie plana • Plaats op een
vlak oppervlak • Posiziona su una superficie
piana • Colocar numa superfície plana

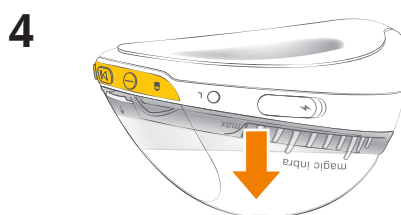


**Milk collects automatically.
Few drops remain in the tunnel.**
Milch wird automatisch
gesammelt. Einige Tropfen
verbleiben im Tunnel.

Le lait est collecté automatiquement. Quelques gouttes restent dans le
tunnel. • La leche se recoge automáticamente. Quedan algunas gotas en
el túnel. • Melk wordt automatisch opgevangen. Enkele druppels blijven in
de tunnel achter. • Il latte si raccoglie automaticamente. Alcune gocce
restano nel tunnel. • O leite é recolhido automaticamente. Algumas gotas
podem ficar no túnel.

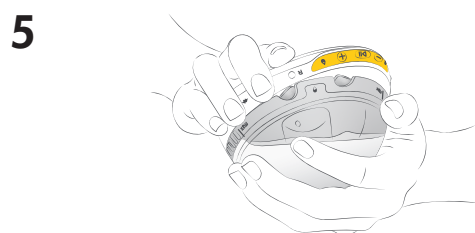

Disassemble in a horizontal position • In horizontaler
Position auseinanderbauen • Démontez en position
horizontale • Desmontar en posición horizontal •
Demonteer in horizontale positie • Smonta in posizione
orizzontale • Desmontar em posição horizontal

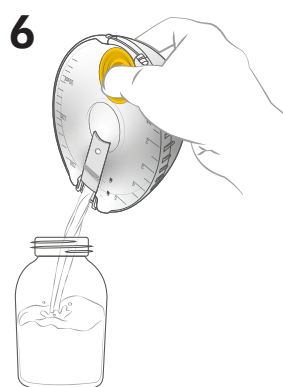

Drops flow into bottle

Tropfen fließen in die Flasche
Les gouttes s'écoulent dans le
contenant
Las gotas caen en el biberón
Druppels uitgieten in het flesje
Le gocce scorrono nella bottiglia
As gotas escorrem para o biberão